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Instructions for operation and maintenance
Scooter for seniors ST-05R black with roof
Ord. number ST-05R

Importer: Steen QOS, s.r.o., Vojkovice 58, Vojkovice. Czech Republic
PLASTIC/CARDBOARD PACKAGING KEEP OUT OF REACH OF CHILDREN, RISK OF
SUFFOCATION

Note:

Read and note instructions and follow the safety instructions and operating instructions
before using the tool.

This product is for hobby and home improvement and it is not professional series!

Original instructions for use.

PERFORMANCE

Your new scooter for seniors will surely satisfy your expectations. It is manufactured to the highest quality standards to meet high criteria for reliable and efficient operation.

You will find that your new scooter is easy to operate and provides safe operation. And if you give it the right care, it will ensure reliable operation for a long time.

Manufacturer: see declaration of conformity

NOTICE

Please read this Operation and Maintenance Manual carefully before using your new product. Pay special attention to all WARNINGS and WARNINGS.

Your product has many features that make your work easier and faster. In the development of this tool, great attention has been paid to safety, operation and reliability with regard to simple maintenance and operation.

TECHNICAL SPECIFICATIONS

motor:	450W/48V
battery:	48V (4x12V), 20Ah Lead-acid (gel)
charging time:	4-6 hours
number of charging cycles:	700
max speed:	25 km
max. load:	200 kg
maximum range per charge:	40 km
max climb rate:	8°
tire size	14"(airless tires)
rim size:	8"
braking system:	drum brake
rear view mirrors	
the wheels have a shock absorbing function	
shopping cart	
adjustable seat and backrest	
chassis material:	steel
suspension axle	
product weight:	125 kg
product dimensions:	1580x730x1200 mm
LED display	
trolley can be used for 2 persons (however the maximum load must not be exceeded)	
white roof	

DESCRIPTION OF CONTROLS

USING THE HANDBRAKE

Press the left brake on the handlebar. 1 and lock it while pressing the foot brake pedal (Fig.2).

Read this user manual carefully before use and keep it for future reference.

The ST-05 is a highly stylish, environmentally friendly trolley with the following advanced design components:

- highly efficient engine with high torque
- controller with battery indicator and voltage protection
- brake system with power cut-off for safe driving
- High performance battery

DRIVING SAFETY

- Like any other product and activity (eg: cycling), the truck poses a risk of injury and damage. By using the product, the user assumes full responsibility for these risks. You must know and observe the driving safety and its risks.
- Your insurance may not cover the accidents associated with this truck.
- The truck is not intended for use on public roads.

SAFETY BASICS

- Before driving, always carry out a safety check and familiarize yourself with the product controls.
- Wear an approved safety helmet when driving. Follow the helmet manufacturer's instructions when handling the helmet.
- Keep body parts or objects (loose clothing, etc.) away from rotating parts of the product. Doing so may cause the product to get caught and damage or damage it.
- Wear suitable footwear when driving to allow full pedaling.
- When driving, wear clear and visible clothing that is not loose to prevent it from getting caught in moving parts of the product.
- When driving, maintain speed in accordance with safe terrain and safe operating conditions.
- Never drive with headphones in your ear that can distract you from focusing and cause confusion about other sounds. Headphones can also get caught in moving parts of the product and cause loss of control.
- Never overload the product. Observe the maximum load specified by the manufacturer.
- Never jump on the product as this may damage the product and cause injury.
- Never use the product if anything would obstruct your view of the environment or risk losing

control of the product.

- Never tow or hold onto another vehicle.
- Never use the product if you are tired, sick or under the influence of addictive substances and alcohol.
- Avoid driving in bad weather, hidden visibility, dusk or in the dark, or if you are tired, if possible. Each of these situations increases the risk of an accident.
- Never allow children to use the product. The product may not be used by persons who are not sufficiently familiar with the safety instructions of the product and this instruction manual.

OPERATIONS AND RECOMMENDATIONS

- Read these operating instructions carefully before driving.
 - When using the industrial truck, observe all rules and regulations in your area.
 - Drive predictably.
 - Use the signal lights to indicate turning or stopping.
- Concentrate on the way forward. Avoid driving on gravel, dirt, wet roads, oil-stained roads, curbs, grilles and other obstructions.
- Beware of unexpected events, such as opening car doors or leaving cars from hidden exits.
 - Familiarize yourself with all the functions of the truck. Use signal lights and other safety and signaling elements.
 - Do not transport anyone in front of you. Doing so may impair the view from the truck.
 - Do not use any objects that may impair your hearing.
 - Maintain sufficient braking distance from all objects. The safe braking distance depends on weather and road conditions.

WRONG WEATHER (RAIN OR WET)

- Bad weather worsens braking and the risk of an accident increases. This may make it difficult to control the truck and lose control more easily. To ensure safe driving, adjust the road speed and weather conditions and brake earlier and gradually than under normal conditions. Reduce driving speed, avoid sudden braking, and drive corners with extreme caution. Always wear reflective clothing and safety lights when driving in poor visual conditions. The surface is slippery in wet terrain, so follow all these instructions.

RIDING AT NIGHT OR DARK

Visibility is poor and impaired at night. If you have to drive under these conditions, check and make sure you comply with your local night driving regulations. Observe the following precautions:

- Make sure your truck is equipped with correctly positioned and securely mounted reflectors.
- Make sure that your clothing or cargo (in the basket) does not obstruct the reflector or light, and always make sure that all lighting is fully functional before driving.
- Wear reflective clothing and reflective elements.
- Drive slowly and avoid hazards such as potholes and curbs.
- Avoid dark, unlit areas and reduce speed. If possible, drive only on well-known routes with good lighting.

OTHER WARNINGS

- This wheelchair is designed to perform best on level ground. Use caution and caution when descending. Repeated acceleration / braking causes the battery to discharge more quickly.
- The truck is equipped with an electric brake system. When you brake, the throttle stops working. Release the throttle before braking.
- Recharge the battery regularly when the truck is stored to avoid undercharging the battery. Charge the battery at least once every 30 days for storage.
- Do not store the truck, battery or charger in direct sunlight. Store it in a dry place away from unauthorized persons. Exposing the truck to rain or snow or other weather conditions can damage the truck.

IMPORTANT

The electric wheelchair is powered by four 12V batteries connected in series to provide enough 48V. All batteries are charged simultaneously. Charge the truck after each journey. When stored, the battery should be recharged within 30 days.

WARNING!

ANY INTERACTION IN THE ELECTRICAL INSTALLATION OF THE PRODUCT IS strictly forbidden !!!

**THE PRODUCT IS SUBJECT TO 48 V ELECTRICAL VOLTAGE WHEN INTERVENTING THE PRODUCT
MAY INJURY THE OPERATOR!**

WARNING

Keep the battery and charger away from heat or fire and flammable substances. Do not expose the battery or charger to rain or moisture.

- Strict prohibition of battery handling. Don't take it out!
- The supplied battery can only be charged with the original charger supplied with the product. The maximum charging current is 4A.

OTHER MAINTENANCE

- If the unit is damaged or incomplete, do not repair or use the unit. Contact your dealer or an authorized service center. Improper interference with the product may cause damage to the product. Any intervention or modification to the truck will void the warranty and void the warranty.
- Your trolley should be properly adjusted once a year and should be thoroughly inspected.
 - Front and rear brakes
- The front brake setting is the same as for a conventional motorcycle. It is very important to brake effectively and make sure it is in good condition. After releasing the brake lever, the wheel should rotate without resistance. If it is one third of the original thickness - the brake shoe should be replaced.
- The rear drum brake is very similar to electric scooters. If it is worn, the whistle will come out of the brake when the brake is applied.

- Front and rear tires

- The spare tire replacement method is the same as for a conventional wheel. Rear tire replacement is more difficult due to engine and brake assemblies and associated wiring and wiring. If you are not sure about the replacement, contact your dealer.

CLEANING

- 1) Never wash the truck with high pressure water to avoid damage to electrical components.
- 2) Never use grease to clean metal wheel parts and never use drum brake grease and tires.

SCHEDULED MAINTENANCE

ST-05 should be checked from time to time.

Refer to the maintenance schedule in the following chart:

Check • adjust ♦ Change ▲ Lubricate ■

Parts	daily	30 days	180 days
Bolts, nuts, screws etc, to insure that nothing is loose	•	• ♦	• ■
Foot board, to ensure It is not worn out	•	• ■	• ■
Tire pressure, to ensure good tire pressure	•		
Drive equipment, to ensure good position; not too loose, not too tight	•	• ♦	• ■
Brake pads, to ensure it is not worn out		• ♦	• ♦
Wheel rim, to ensure it is not distorted			
Steering Linkage, to ensure it is not distorted			• ■
Front light, bugle, to ensure functioning properly	•	•	
Brake, to ensure it is working properly	•	• ♦	• ♦
Throttle, to ensure that it is working property	•		
Steering system, to ensure operation is	•	■	
Wheel axle, to ensure if it is loose	•		■
Motor	•	■	

TROUBLESHOOTING GUIDE

NOTE :If other problems occur, do not attempt to disassemble tricycle on your own, please proceed to the tricycle dealer to call Customer Service.

Problem	—Cause	Solution
Battery gauge doesn't register anything when, vehicle is ON	a) Loose connection near the power switch or battery pack. b) Battery is completely discharged.	a) Check the connections. Reconnecting any that are loose. b) Charge the battery.
Battery gauge register but vehicle still does not work	a) Motor wire is loose or disconnected. b) Throttle is defective. c) Control is defective	a) Send to service to repair the wiring. b) Replace throttle. c) Replace controller.
Noise coming from motor	Magnets in motor have come loose.	Send to after service to replace a new motor.
Vehicle does not stop when throttle returns to original position	Controller or throttle is defective	Replace controller or throttle. Customer service can assist in diagnosis.
Vehicle is losing speed	a) Battery is low. b) defective battery. c) defective throttle.	a) Charge battery. b) Replace battery. c) Replace throttle.
Distance per charge is reduced	a) Tire pressure is low. b) Battery is not fully charge. c) Battery is damage. d) Headwind, slopes, overloading, etc.	a) Fill tires with sufficient pressure. b) Charge battery for 6-8 hours. c) Replace battery. d) Reduce these factors.
Battery could be charge	a) Loose wiring near the charging socket. b) Loose wiring and battery pack. c) defective charger.	a) Check the wiring, tight connection. b) Send to a qualified service technician. c) Replace charge.

Warranty Sharks

Warranty certificate belongs to the product sold corresponding number as its integral part. The loss of the original warranty card will be grounds for non-recognition of warranty repairs. On the Products is warranty period for the customer (private person) 24 months from the date of purchase. For a business customer is warranty period 12 months from the date of purchase.

During the warranty period Sharks authorized service charge will eliminate any product defects due to manufacturing defects or faulty material.

The manufacturer guarantees the preservation of adequate functional properties of the tool and its safe operation only in case of realization of servicing tool Dealership.

The guarantee does not cover:

- Components that are subject to natural wear and tear caused by use in accordance with operating instructions
- Defects in the tool caused by non-compliance with the operating instructions
- Tools to which changes or additions have been made.
- Defects in the tool caused by instructions, improper use, abnormal environmental conditions, inappropriate operating conditions, overload or insufficient servicing or maintenance.

The guarantee submit to the retailer or to the authorised service Sharks in original package and with the copy sales receipt and completed complaints card.

After receipt complete product (in folded state) service assess legitimate complaint.

Possible warranty repair service will confirm to the warranty certificate, which is part of this manual.

The warranty period is extended for the period from receipt of the complaint service (dealer) after handing over the repaired product back to the customer. Importer reserves 30 days to assess the claim in terms of entitlement to free repair (replacement) under warranty. When selling is the seller obligated to inform the buyer Valet product, showcase product and properly fill out the warranty card.

Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your Local Authority or retailer for recycling advice.

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We:

Wuxi Everbright Co.,Ltd.
NO.20,Middle Dongting Road,Wuxi,Jiangsu,China

declare on their sole responsibility that

Product: Scooter for seniors ST-05R black with a roof
Type: **ST-05R (ST-05)**

Equipment designed for private use to which this declaration relates, is under common conditions safe and complies with the following technical regulations:
The following regulations also apply to battery charger type: YX-180W and battery type: 6-DZM-12

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WUXI EVERBRIGHT CO., LTD.